
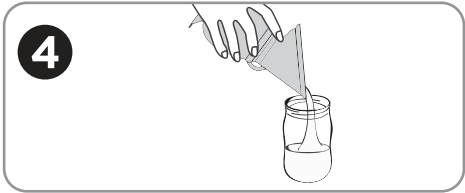
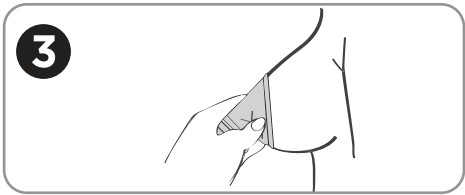
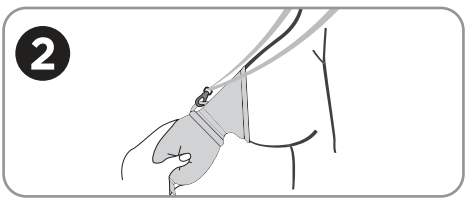
# Lansinoh®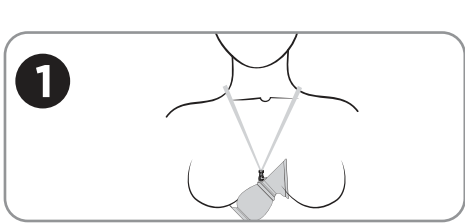

## BREASTMILK COLLECTOR

**GB** INSTRUCTIONS FOR USE

It’s common for one breast to leak when feeding or pumping from the other side and we think this is such a waste.

The Lansinoh® Breastmilk Collector fits securely to the breast creating natural suction. It offers a simple way to collect precious breastmilk, so your little one can enjoy every last drop.

#### DIRECTIONS FOR USE

**Step 1** Before feeding or pumping, attach the neck strap to the Breastmilk Collector and place around your neck to prevent spills later on.

**Step 2** Squeeze the centre of the Breastmilk Collector and place over your breast to form an airtight seal. Breastmilk will be collected once your let-down begins.

**Step 3** Once your collector is full or you have stopped feeding, release by gently squeezing the flange area while securely holding the base.

**Step 4** Pour your expressed breastmilk into a Lansinoh® Breastmilk Storage Bag or Breastmilk Storage Bottle. Place cap over the top of your Collector to prevent contamination.

#### TIPS

Because of the natural way that your body releases breastmilk, the amount of fat in the milk available to baby gradually changes as a feed progresses. We suggest that you mix breastmilk collected with your Lansinoh® Breastmilk Collector with expressed breastmilk to ensure the maximum nutritional benefits when feeding your little one.

If you are struggling to get your milk flow going, why not try a Lansinoh® Thera®Pearl® 3-in-1 Breast Therapy to encourage let down and promote faster milk flow.

#### CLEANING & CARE

Follow these guidelines unless specifically advised otherwise by your healthcare provider.

Before first use, place your Breastmilk Collector in boiling water for 5 minutes, allow to cool, thoroughly rinse and air dry.

- Always wash all components of the product immediately after each use.
- Can be washed in hot soapy water and rinsed in clean water.
- Dishwasher safe on top rack only.
- Safe to clean in a microwave–safe, steam sterilizer bag and special sterilizing machine.

Remove neck strap prior to cleaning. Do not clean with solvents or harsh chemicals. Avoid contact with sharp objects

**WARNING**  
**THIS PRODUCT IS NOT A TOY. The Breastmilk Collector neck strap is solely designed and intended for wear by adults. Do not place the neck strap around baby’s neck. Babies and children should never be left unattended with the Breastmilk Collector neck strap. Do not let baby chew on clasp. Adult supervision is always required. Discard if damaged.**

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